**en****Dartin Select 2000 ml Gravity Drainage Bag****INTENDED USE:**

For collecting drainage fluids

**DIRECTIONS FOR USE:**

Visually inspect the product for any imperfections or surface deterioration prior to use.

1. Wash hands
2. Connect to Catheter
3. Position tubing to ensure an unimpeded flow down to the urine bag. Hang drainage bag on bed frame.
4. Secure tubing to sheet with bed sheet clamp.
5. To empty bag:
  - a. Wash hands.
  - b. Remove drain tube from holder.
  - c. Release clamp and empty bag into collection vessel.
  - d. After emptying the bag, close clamp before placing in the folder.
  - e. Wash hands.
6. Periodic observations of this system should be made to secure urine is flowing freely. If a standing column of urine is observed, check for correct positioning of bag and tubing and then for physical obstruction. If correct positioning or removal of obstruction does not allow free flow, the bag may need to be changed.
7. Wash hands.

**STERILE EO**

Sterile if package is unopened and undamaged. For single use only. The reuse of this single-use device can lead to patient infection and/or device malfunction.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

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