


# EN Dartin Bilateral Reduction/Night Kit

## Intended Use:

For collecting drainage fluids from two catheters into one bag. Provide simple exchange from Day Bag to Night Bag.

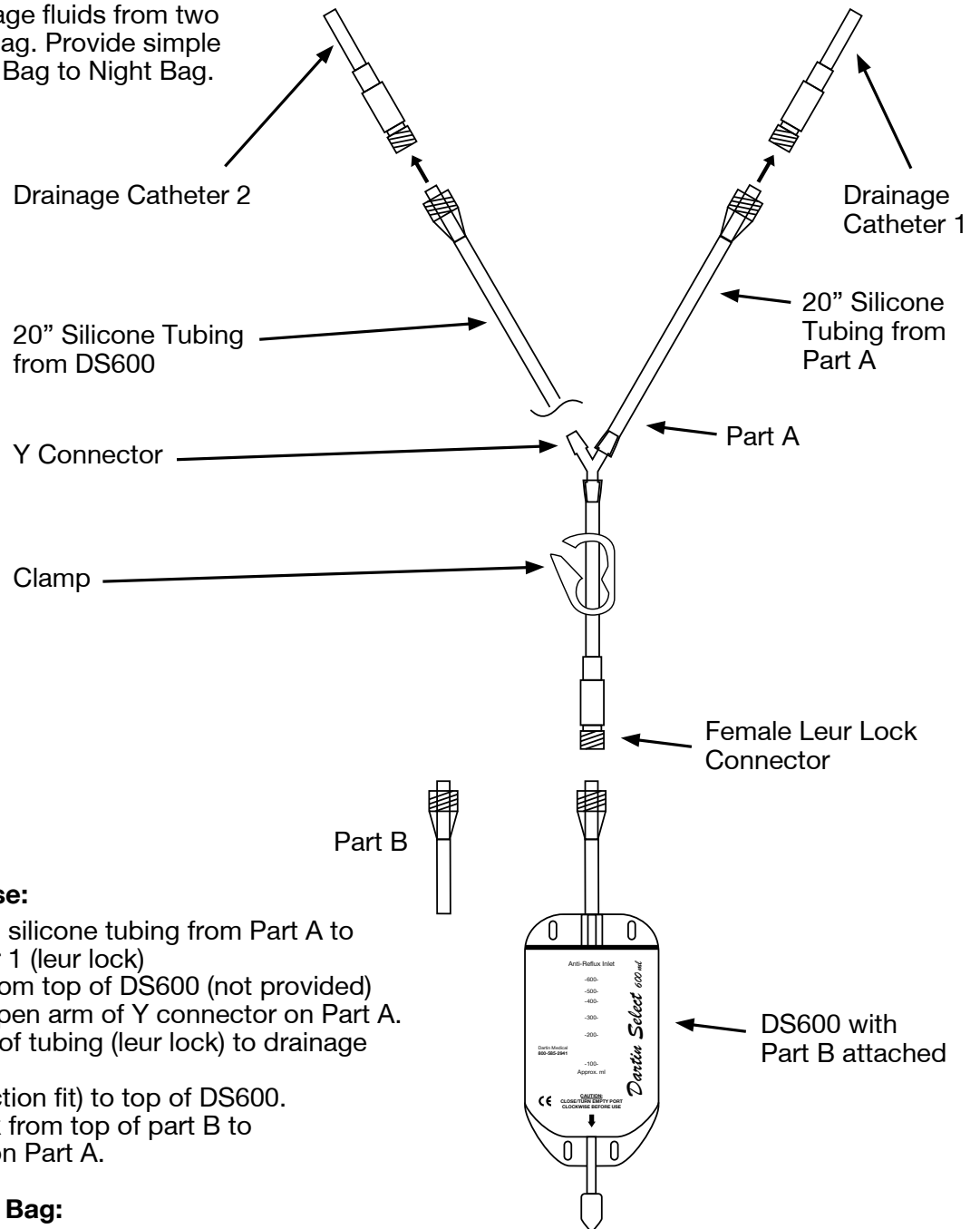

## Instructions For Use:

- 1 Attach end of 20" silicone tubing from Part A to drainage catheter 1 (leuc lock)
- 2 Remove tubing from top of DS600 (not provided) and reattach to open arm of Y connector on Part A. Attach other end of tubing (leuc lock) to drainage catheter 2.
- 3 Attach Part B (friction fit) to top of DS600.
- 4 Connect leuc lock from top of part B to female leuc lock on Part A.

## To Switch to Night Bag:

- 1 Clamp off tubing at bottom of Part A. Using proper sterile technique, undo connection between Part A and Part B.
- 2 Attach tubing (leuc lock) from DNLL2000 Night Bag to Part A.
- 3 Release clamp on Part A.

**STERILE EO**

Sterile if package is unopened and undamaged. For single use only.

The reuse of this single-use device can lead to patient infection and/or device malfunction.

**Caution** Federal Law (USA) restricts this device to sale by or on the order of a physician.


**Dartin Medical Systems Canada**

**To Reorder:** Call 1-800-585-2941 Fax: (905) 403-8631

# FR Dartin Bilateral Reduction/Kit Nuit

## Employ Prévu :

Pour la collecte des fluides de drainage de deux cathéters dans un seul sac.  
Fournir un échange simple du sac de jour au sac de nuit.

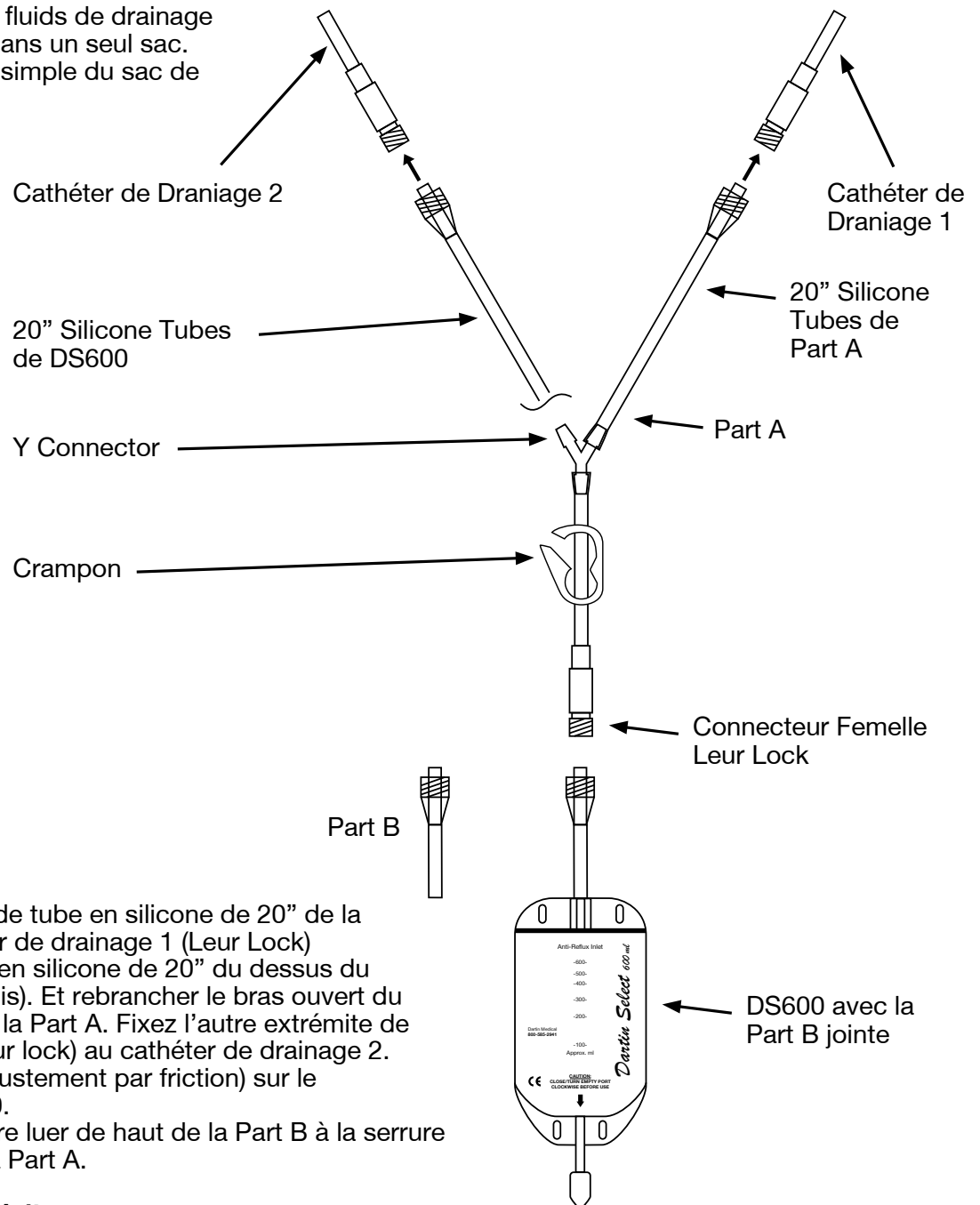

## Mode D'Emploi :

- 1 Fixez l'extrémité de tube en silicone de 20" de la Part A au cathéter de drainage 1 (Leur Lock)
- 2 Retirez les tubes en silicone de 20" du dessus du DS600 (non fournis). Et rebrancher le bras ouvert du connecteur Y sur la Part A. Fixez l'autre extrémité de cette tubulure (leur lock) au cathéter de drainage 2.
- 3 Fixez la Part B (ajustement par friction) sur le dessus du DS600.
- 4 Brancher la serrure luer de haut de la Part B à la serrure leur femelle sur la Part A.

## Passer au Sac de Nuit :

- 1 Fixer le tube au bas de la Part A. En utilisant une technique aseptique, annule la connexion entre la Part A et la Part B.
- 2 Fixez la tubulure (leur lock) du Sac de Nuit DNLL2000 à la Part A.
- 3 Relâcher la pince sur la Part A.

**STERILE** **EO** 

Stérile si l'emballage n'est pas ouvert ou endommagé.  
Stictement à usage unique, si ce dispositif à jeter après usage est réutilisé, cela pout conduire à l'infection du partient et/ou la défalliance du dipositif.